


CRANIAL AND SPINE SOLUTIONS





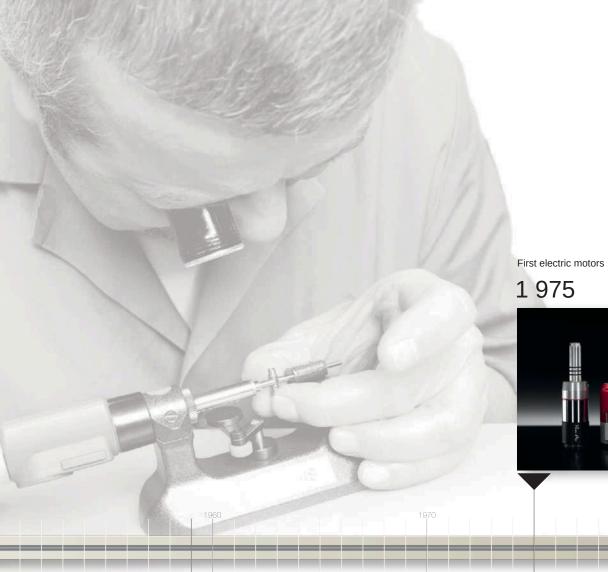



David Mosimann founded Bien-Air in 1959. His son, Vincent, currently presides over the Bien-Air Medical Technologies Group.

## **BIEN-AIR** CREATOR OF EXCELLENCE **SINCE 1959**

ORL-E-92

with Professor Fisch

1 992



The legendary pedal developed









2007 OSSEODUO The high-speed multifunction control unit

2014

Bien-Air reinvent the handpiece. Ergonomic design and state-of-the-art technology



1 959

The Bien-Air adventure begins in Neuchâtel



1 981

RHEOSTAT First milling system for ear surgery

Offering reliable and easy to use instruments A true culture of excellence, which fits to simplify the work of practitioners and perfectly with the tradition of Swiss-Made constantly improve patient comfort has been products from the renowned Watch Valley. Bien-Air's mis- sion since its creation in 1959. Bien-Air supplies high quality products, which

and dentistry, Bien-Air has won global recog- processes. nition for the performance and reliability of its Bien-Air is a human-scale family business. Its p ro d u cts.

the core of the development of every new personalised attention. product. Constantly in tune with the day-to- Our promise is to continue our innovative day require- ments of practitioners, Bien-Air approach in order to provide you with has moved beyond its previous innovation advanced instruments and solutions. Making boundaries, con- stantly setting the bar your work easier, every day. higher.

can only be guar- anteed by using state-of-

With over 60 years' experience in ENT surgerythe-art technologies and manufacturing

close relationships with its customers and partners are one of its most valuable assets. Ergonomics, precision and durability are at This is why every individual receives

Ultra-precision control of motors and handpieces

# OSSEODUO SIMPLE AND PRACTICAL

NANO 8

NANO MICROMOTOR

The NANO micromotor is

compact, lightweight and

extremely quiet. It provides

torque.

80,000 rpm

•0000

80g





Max speed: Sound level: Weight: RAPIDO MICROMOTOR 80,000 rpm

115g

procedures.

The RAPIDO is a powerful, ventilated micromotor with excellent torque. It provides significant cutting performance even in challenging surgical situations. The directional self-ventilation provides low operating temperatures, consistent performance and comfort during long

n challenging surgical ergonomics, and less fatigue at the end of each surgical procedure. Ing temperatures, tent performance

RAPIDO and NANO micromotors are compatible with the range of Bien-Air PM2 handpieces.

High-speed shaping up to 80,000 rpm


Integrated irrigation
with adjustable flow rate
between 8 and 135 ml/min

With its convenient size and versatility the OSSEODUO paves the way for an optimal

surgical experience.

Developed by and for surgeons, the OSSEO- DUO combines simplicity of use with perfor- mance to provide a comprehensive portfolio of micromotors and instruments. One OSSEODUO feature allows you to rapidly select any connected micromotor from the console or the foot page 1.0SSEODUO is a multi-functional power console specifically designed for the following surgical specialties: — ENT Surgery — Neurosurgery — Spine Surgery

Offering independence to the surgeon, the pedal controls the micromotor's speed, rotation direction and activates irrigation.

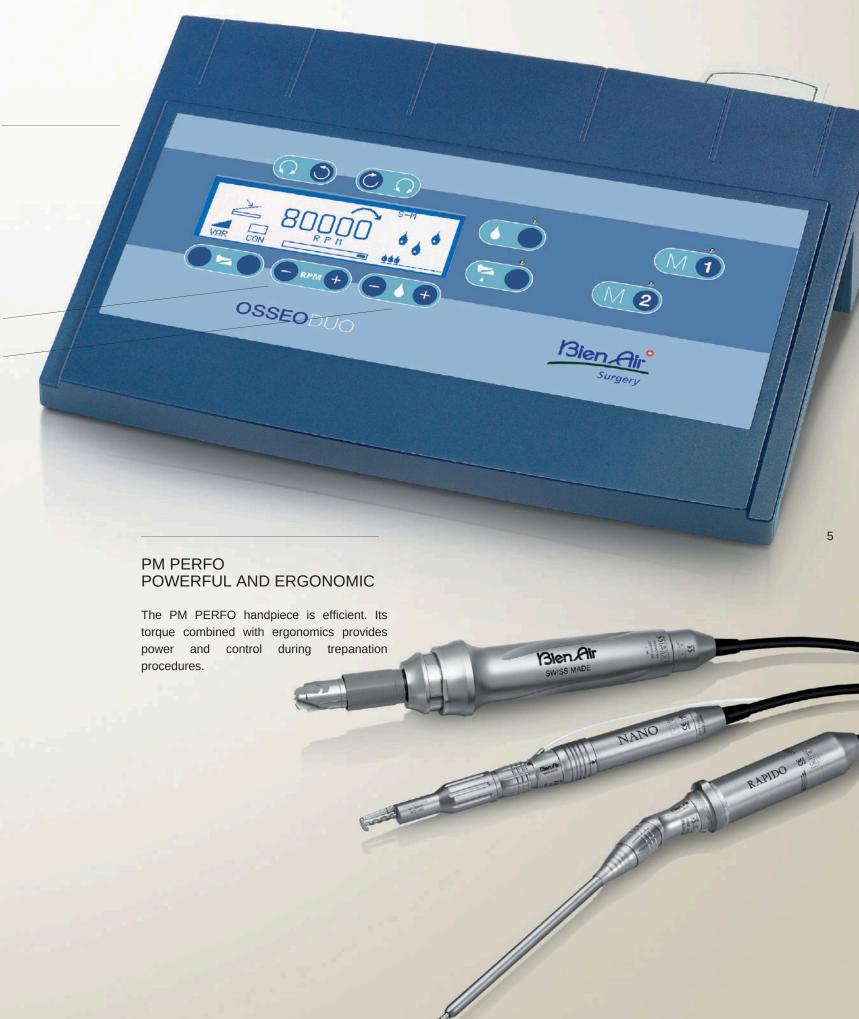




The PM2-S70HD heavy-duty straight hand- piece boasts an ingenious system of detacha- ble guards. This allows you to attach the guard of your choice in an instant.

With the Bien-Air system there is no need for a craniotomy specific handpiece, you can just simply connect a craniotomy guard to your stan- dard straight handpiece.

Thanks to Bien-Air guards, there is no need for manual external irrigation. The guards are equipped with irrigation inside the instrument. Your cutting tool is provided with precise irriga- tion via two nozzles at the end of each guard.



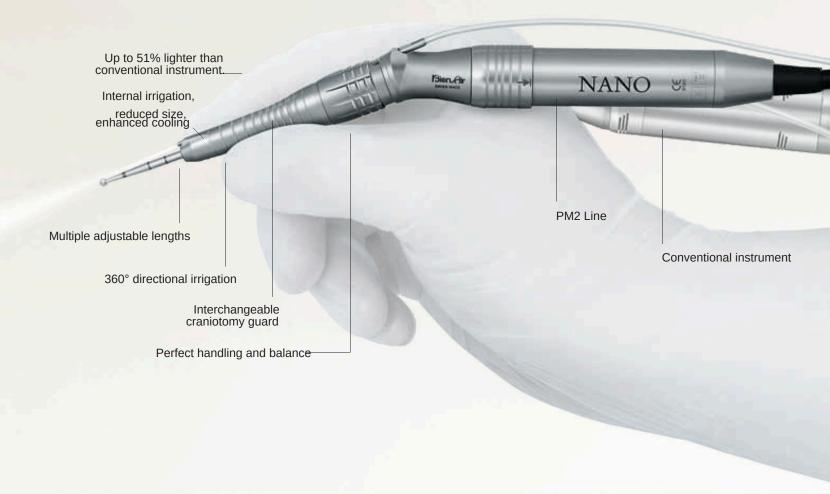

### PM2 HANDPIECES

#### REDEFINING ERGONOMICS

#### **INTERNAL IRRIGATION**

Handling your instrument requires less effortOur heavy duty handpieces feature a leading to greater accuracy in your movementsique irrigation system. The irrigation and less fatigue.

hose connection is at the base of the instrument and is internally channeled

The Bien-Air PM2 standard connection offersthrough to the distal end.

a noticeable reduction in size and a great This design keeps your handpiece from overflexibility. Each PM2 handpiece is compatibleheating during long procedures. with the NANO and RAPIDO micromotors.

Your handpiece will remain comfortable and cool throughout lengthy or demanding surgical case.



The internal irrigation system cools the handpiece while ensuring perfect handling and visibility

#### PM2 AHEAD OF THE FIELD

The Bien-Air PM2 handpieces are specifically built for Spine surgery. The design has been optimized to deliver precision and power, while maintaining the visibility needed when working on delicate structure and confined spaces.

An angle of 20° to maximize accessibility



PM2 handpieces allow 3 bur extension lengths.

### ADJUSTABLE BUR


The PM2 burs are fitted with additional retentive locking grooves at the base. This means that the bur can be extended to 3 different lengths, giving you more access to delicate areas while providing better visibility without compromising bur stability.





The triangular coupling prevents any loss of drive during use and all the motor torque is transferred to the bur for greater performance.

### PRODUCTS OVERVIEW

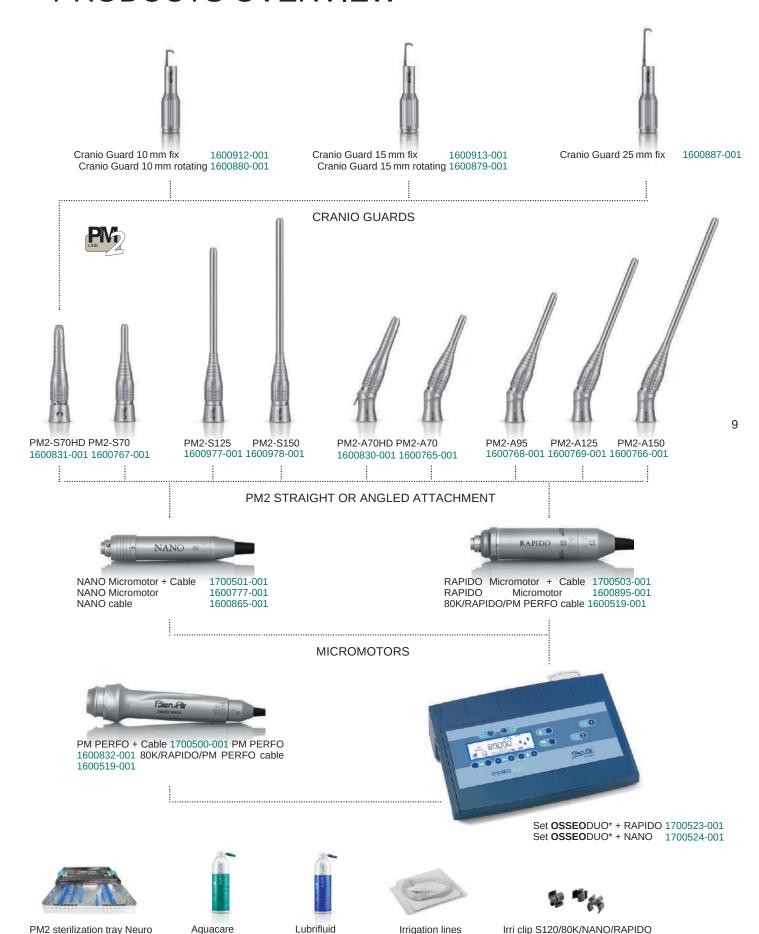

\*Each set includes 1 control unit, 1 foot pedal, 1 irrigation stand, 1 power cable, 10 irrigation lines, and 10 irrigation clips.

1600949-001

## PM2 – 80K\* BURS RANGE



• 70 MM

	STAINLESS STEEL	0-	DIAMOND		CARBIDE	<b>-</b>	FULL CARBIDE
	Sterile – 5 pces / box		Sterile – 1 pce / box		Sterile – 1 pce / box		Sterile – 1 pce / box
ø mm	Single use 1100351-005	ø mm	Single use 1100365-001	ø mm	Single use	ø mm	Single use
0.6	1100351-005	0.6	1100365-001	0.5	1100392-001	0.5	1100475-001
0.8	1100352-005	8.0	1100360-001	0.8	1100393-001	1.0	1100462-001
1.0	1100353-005	1.0	1100367-001	1.0	1100394-001	1.5	1100463-001
1.4	1100354-005	1.4	1100369-001	1.4	1100395-001	2.0	1100464-001
1.8	1100355-005	1.8	1100309-001	1.8	1100396-001	2.3	1100465-001
2.3	1100350-005	2.3	1100370-001	2.3	1100397-001	3.0	1100466-001
2.7	1100357-005	2.7	1100371-001	2.7	1100398-001	3.5	1100483-001
3.1	1100358-005	3.1	1100372-001	3.1	1100399-001	4.0	1100467-001
3.5 4.0	1100355 005	3.5 4.0	1100374-001	3.5	1100400-001	4.5 F.O	1100484-001
4.0	1100360-005	4.0	1100375-001	4.0 4.5	1100401-001	5.0 6.0	1100476-001
5.0	1100361-005	5.0	1100376-001	5.0	1100402-001	7.0	1100468-001
6.0	1100362-005	6.0	1100377-001	6.0	1100403-001	7.0	1100477-001
7.0	1100364-005	7.0	1100378-001	7.0	1100404-001		
7.0	2200001 000		2200010 001	7.0	1100405-001		

COARSE DIAMOND Sterile – 1 pce / box Single use

1100379-001

1100380-001

ø mm

3.1

6.0

2.3

10



6.0

EXTRA COARSE

MATCHSTICK

	Sterile – 1 pce / bo
	Single use
ø mm	1100460-001
1.8	1100461-001
3.0	1100401-001

TWIST DRILL INOX Sterile – 1 pce / box

SPIRAL

	Single use			
ø x L mm	1100459-001			
1.5 x 12	1100433-001			
1.1 x 5	1100473-001			

STRAIGHT CRANIOTOMY • Sterile – 1 pce / box Single use L mm

10

1100383-001

1100384-001

1100414-001





STAINLESS STEEL DIAMOND Sterile – 1 pce / box Sterile – 1 pce / box Single use Single use ø mm ø mm 1100446-001 1100320-001 1.0 1.0 1100447-001 1100321-001 1.8 1.8 1100448-001 1100322-001 2.3 2.3 1100323-001 1100449-001 3.1 3.1 1100450-001 1100324-001 4.0 4.0 1100451-001 5.0 1100452-001

	COARSE DIAMOND Sterile – 1 pce / box		EXTRA COARSE DIAMOND Sterile – 1 pce / box		
	Single use		Single use		
ø mm	1100325-001	ø mm	1100444-001		
3.1	1100326-001	3.8	1100445-001		
4.0		4.7			
5.0	1100327-001	6.0	1100490-001		
6.0	1100328-001	0.0			
0.0					
TARTE TO STATE OF THE STATE OF					

<b>*</b>	DRUM Sterile – 1 pce / box Single use	<u></u>	MATCHSTICK Sterile – 1 pce / box Single use
ø x L mm	Siffgle use	ø mm	1100336-001
5 x 10	stainless steel: 1100339	9-001 1.8	1100337-001
4 x 8	coarse diamond: 11003	38-00 <b>3</b> .0	1100001 001

• 125 MM

	STAINLESS STEEL Sterile – 1 pce / box	0-	DIAMOND Sterile – 1 pce / box		COARSE DIAMOND Sterile – 1 pce / box		EXTRA COARSE DIAMOND Sterile – 1 pce / box
	Single use		Single use		Single use		Single use
ø mm	1100453-001	ø mm	1100308-001	ø mm	1100313-001	ø mm	1100316-001
1.8	1100454-001	1.0	1100309-001	3.1	1100314-001	3.8	1100317-001
2.3	1100454-001	1.8	1100303 001	4.0	1100314-001	4.7	1100491-001
3.1	1100456-001	2.3	1100310-001	5.0	1100313-001	6.0	
4.0		3.1					
5.0	1100457-001	4.0	1100312-001				
6.0	1100458-001						
				-	DRUM Sterile – 1 pce / box	=	MATCHSTICK Sterile – 1 pce / box



	STAINLESS STEEL Sterile – 1 pce / box	<b>O</b> -	DIAMOND Sterile – 1 pce / box
	Single use		Single use
ø mm		ø mm	o o
3.1	1100344-001	1.0	1100329-001
	1100345-001		1100330-001
4.0		1.8	1100331-001
		3.1	
		4.0	1100332-001
		4.0	

-	COARSE DIAMOND Sterile – 1 pce / box			MATCHSTICK Sterile – 1 pce / box Single use
ø mm	Single use 1100333-001		ø mm	1100481-001
3.1	1100333-001		1.8	1100481-001
4.0	1100334-001		3.0	1100402-001
5.0	1100000-001			

Single use

coarse diamond: 1100342-001 1.8

Single use g mm stainless steel: 1100343-001

5 x 10

4 x 8

Single use

1100340-001

1100341-001

11

